

# Advasil® Conform

## Silicone gel sheet for scar management



A sheet of Medical Grade self-adherent silicone gel for the management of scar tissue or recently healed wounds.

Composed of a conformable layer of translucent Silfix® silicone gel and a smooth polyurethane outer layer to improve wear ability and patient compliance.

### Use

Remove Advasil® Conform from the pack, remove the liner and place gel side to the scar. For smaller scars Advasil® Conform can be cut to size. For larger scars use additional sheets of Advasil® Conform and place side by side. Advasil® Conform is self adherent and reusable for up to 28 days.

Advasil® Conform should be washed at least once daily with warm water and mild soap. Avoid using heavy soaps or detergents as they may cause disintegration of the silicone and/or skin irritation. Gently pat the product dry using a lint free cloth. Ensure the product is dry before re-applying to the skin or storing. When storing Advasil conform replace the liner and store in original packaging. It is important that the skin beneath the dressing is kept clean, and dry. Wash area daily.

Wear time should be built up over a period of time to condition the skin after which Advasil conform can be comfortably worn for 24 hours.



### Features

- ✓ Can be shaped to optimise coverage of scar area
- ✓ Conformable
- ✓ Washable

- ✓ Reusable for up to 28 days
- ✓ Self-securing with gentle release

### Indications

- ✓ Hypertrophic & Keloid scars

### ordering information

Size	Stock code	Pack size
10cm x 10cm	CR3845	5
10cm x 15cm	CR3832	5